ES

F-830
Manual de instrucciones
WE-2013-F830-1

CEPILLO ROTATIVO

ADVERTENCIA

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ROTARY BRUSH

WARNING

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

EC-DECLARATION OF COMPLIANCE

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2006/95/EC
2. Conditions set out in the Electromagnetic Compatibility Directive: 2004/108/EC

RECYCLING

 Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

WELCOME!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to





incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.



PACKAGE CONTENTS

- These components are subject to change without prior notice.
- Appearance of product may differ from the image.

No.	PRODUCT	REF.	QTY.
1	BRUSH HANDLE	10074-8	1
2	BRUSH HEAD 1	10073	1
3	BRUSH HEAD 2		1
4	BRUSH HEAD 3		1
5	BRUSH HEAD 4		1
6	BRUSH HEAD 5		1
7	MAIN UNIT	-	1
8	FEEDER CABLE	-	1

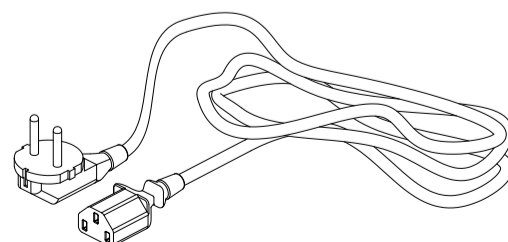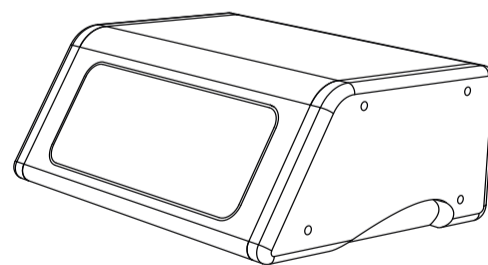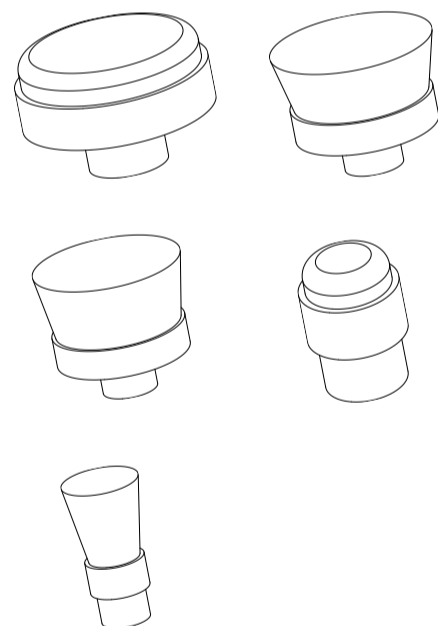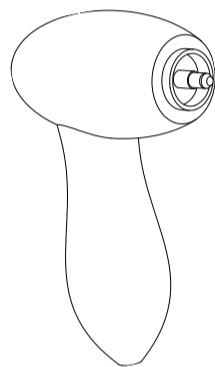FRONT PANEL/REAR PANEL

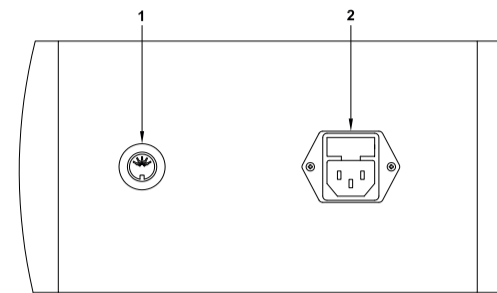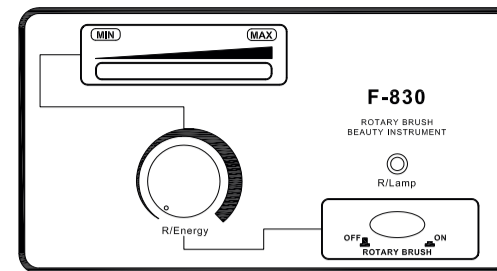1. Brush handle
2. Feeder cable


INSTRUCTIONS OF USE

BEFORE TREATMENT:

1. Scrub the skin surface with a wet pad.
2. Using a steamer (or hot cloth) to soften the skin surface in the treatment area.
3. After skin-cleansing maintenance, it is advisable to daub some nutritional oil on the target skin surface.

STARTING TREATMENT:

1. Connect the power supply via the rear panel, indicated in the sketch as "POWER LINE INPUT".
2. Connect the accessories properly to the front/rear panel as indicated in the sketch.
3. Choose the appropriate brush head according to the customer's skin type. In general, delicate or dry skin needs a soft brush head, while harder brushes can be used for rough or greasy skin.
4. Turn on "ROTARY BRUSH".
5. While holding the brush handgrip, adjust "R/ENERGY". The speed of rotation will increase as the adjusting angle is increased.

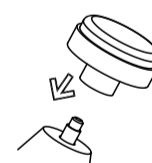CAUTIONS

- Do not press the brush head excessively while it is running.
- Do not operate this instrument with any evident acid or alkaline substance.
- After the operation, turn the instrument off and disconnect the power cord.
- This function should not be used on customers who suffer from hypersensitive skin or dermatitis.
- If the capillary vessels in the skin are injured, do not cleanse the skin with this function.

MAINTENANCE

- After the operation, submerge the brush head in clear water with a little cleanser for 5 minutes. Then, after rinsing it, submerge it in alcohol for half an hour.
- The brush head must be disinfected using ultraviolet radiation and then kept in its original packaging.

TECHNICAL SHEET

Weights and sizes	Packing size (cm)	36x31x18,5
	Gross weight (kg)	4,3
	Product size (cm)	31x17/24x13
Working characteristics	Speed of rotation	0 - 550 RPM
Electrical characteristics	Rated voltage	220V ~ 240V
	Frequency	50Hz/60Hz
	Rated power	25 W

ACCESSORIES

REF.	DESCRIPTION
10074-8	BRUSH HANDLE
10073	BRUSH HEAD 1
	BRUSH HEAD 2
	BRUSH HEAD 3
	BRUSH HEAD 4
	BRUSH HEAD 5

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